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REF K079

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FINALIDADE

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1. Thompson D, Milford-Ward A, Whicher JT. The value of acute phase protein measurements in clinical practice. Ann Clin Biochem 1992;29:123-31.
2. Gabay C, Kushner I. Acute-phase proteins and other systemic responses to inflammation. N Engl J Med 1999;340:448-54.
3. Hansson LO, Lindquist L. C-reactive protein: its role in the diagnosis and follow-up of infectious diseases. Curr Opin Infect Diseases 1997; 10:196-201.
4. Sipe JD. Acute-phase proteins in osteoarthritis. Semin Arthritis Rheum 1995;25:75-86.
5. Dati F, Schumann G, Thomas L, Aguzzi F, Baudner S, Bienvenu J et al. Consensus of a group of professional societies and diagnostic companies on guidelines for interim reference ranges for 14 proteins in serum based on the standardization against the IFCC/BCR/CAP reference material (CRM 470). Eur J Clin Chem Clin Biochem 1996;34:517-20.
6. Schlebusch H, Liappis N, Klein G. High sensitive CRP and creatinine: reference intervals from infancy to childhood. Poster presented at AACCC/SCCC; July/August 2001, Chicago, Illinois.

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1. Thompson D, Milford-Ward A, Whicher JT. The value of acute phase protein measurements in clinical practice. Ann Clin Biochem 1992;29:123-31.
2. Gabay C, Kushner I. Acute-phase proteins and other systemic responses to inflammation. N Engl J Med 1999;340:448-54.
3. Hansson LO, Lindquist L. C-reactive protein: its role in the diagnosis and follow-up of infectious diseases. Curr Opin Infect Diseases 1997; 10:196-201.
4. Sipe JD. Acute-phase proteins in osteoarthritis. Semin Arthritis Rheum 1995;25:75-86.
5. Dati F, Schumann G, Thomas L, Aguzzi F, Baudner S, Bienvenu J et al. Consensus of a group of professional societies and diagnostic companies on guidelines for interim reference ranges for 14 proteins in serum based on the standardization against the IFCC/BCR/CAP reference material (CRM 470). Eur J Clin Chem Clin Biochem 1996;34:517-20.
6. Schlebusch H, Liappis N, Klein G. High sensitive CRP and creatinine: reference intervals from infancy to childhood. Poster presented at AACC/SCSC; July/August 2001, Chicago, Illinois.

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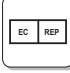
C REACTIVE PROTEIN ULTRASENSITIVE

REF K079

USAGE INSTRUCTIONS

FUNCTION

Method for quantitative determination of C-Reactive Protein (CRP) in very low concentrations. Immunoturbidimetric test, for in-vitro diagnostic only.

PRINCIPLE OF ACTION

Methodology: Immunoturbidimetry

The reagent allows the quantitative determination of CRP in human serum by antigen-antibody reaction. In the presence of a polymer activator, which increases sensitivity and speed of the assay, C-Reactive Protein forms an insoluble complex with specific antibody, causing turbidity, whose intensity increases proportional to the concentration of CRP in the sample.

REAGENTS

Number 1 - Buffer - Store between 2 and 8°C. Contains: HEPES (pH 7,2) 10 mmol/L, Polyethylene Glycol (PEG), NaCl, Sodium Azide 15,38 mmol/L, detergents and stabilizers.

Number 2 - CRP Latex - Store between 2 and 8°C. Contains: Borate Buffer 3,2 mmol/L, Monoclonal and Polyclonal Antibody anti-CRP bound to particles of Polystyrene, Polyethylene Glycol (PEG), NaCl, Sodium Azide 15,38 mmol/L, detergents and stabilizers.

Number 3 - Calibrator - Store between 2 and 8°C. Contains: C-Reactive Protein and Sodium Azide 15,38 mmol/L.

Warning: Concentrations of C Reactive Protein may vary according to the lot - See bottle label.

PRESENTATION

Reagent	Presentation		
	1	2	3
Nº 1	10 mL	25 mL	2 x 2,5 mL
Nº 2	10 mL	25 mL	2 x 2,5 mL
Nº 3	1,0 mL	1,0 mL	2 x 1,0 mL

EQUIPMENTS AND OPERATIONAL INPUTS

This kit is for specific usage in automated equipments.

TRANSPORTATION AND STORAGE CONDITIONS

The storage temperature should be between 2 to 8°C. The transport at temperatures between 15 and 30°C should not exceed 72 (seventy two) hours. Protect from light and avoid moisture. **Do not freeze.**

SPECIAL CARE

- 1- For professional *in vitro* diagnostic use only.**
- Strictly follow the methodology proposed to obtain exact results.
- Water used in material cleaning must be recent and free of contaminants.
- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers, that may alter the results significantly.
- Handle with care, all the reagent that contains Sodium Azide, which is irritant to skin and mucous.
- Reagents 2 and 3 should be handled cautiously since they are subject to biological contamination.
- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Advisory Service Customer) of Quibasa.
- Do not use the product in case of damaged packaging.
- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

SAMPLES

Serum or plasma (heparin or EDTA), free of hemolysis and intense lipemia. Samples are stable in temperatures between 2 and 8°C for up to 8 days and 03 months between -20°C.

PREPARATION OF WORK REAGENT

Reagents are ready for use for double reagents programming. For single reagent programming, mix 1 part of reagent N° 1 with 1 part of reagent N° 2, previously homogenized. The reagent must be prepared just before the moment of its usage. Verify programming.

TECHNIQUE

For calibration of the reaction, utilize preferentially the kit's internal calibrator or the Bioclin Multical kit. For control use the Bioclin Multicontrol Kit.

This kit should only be used in automated equipment. Check programming through the website www.bioclin.com.br or through the SAC: 0800 031 5454.

PROCESS LIMITATIONS

When the CRP value is less than 1,0 mg/L, triglycerides up to 2000 mg/dL did not influence the results. Bilirubin above 40 mg/dL, and hemoglobin greater than 1000 mg/dL significantly affect the results.

INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control, where all procedures, rules, limits and tolerance to variations be clearly established. It is important to mention that all measurement systems present an analytical variety, and it must be monitor by the laboratory. Therefore, it is recommendable the use of controls, allowing the precision and accuracy of the dosages.

TRACEABILITY

The kit's calibrator is traceable to the reference material NIBSC 85/506 (Human C-Reactive Protein 1st International Standard - WHO International Standard).

REFERENCE VALUES

The reference values for this method were obtained through the determination of C-Reactive Protein in healthy populations of male and female. Values are indicators of cardiovascular risk:

- High Risk: greater than 3,0 mg/L
- Medium Risk: 1,0 to 3,0 mg/L
- Low Risk: less than 1,0 mg/L

IMPORTANT

- The interpretation of values for cardiovascular risk can only be done in the absence of systemic inflammatory conditions, since their evaluation is also a parameter for definition of inflammatory conditions.
- CRP should not be used in isolation but must be associated with clinical test or complementary exams.
- These values should be used as guidelines, each laboratory should establish its range of reference values, according to the population served.
- The results provided by this kit must be interpreted by the medical professional responsible, not being the only criterion for the determination of diagnosis and/or treatment of the patient.

PRODUCT PERFORMANCE

QUALITY CONTROL

Accuracy

RECOVERY

The recovery analysis was performed with 05 determinations of samples. Accuracies were calculated and were found in good agreement with the reference values, obtaining a recovery between 94 and 107%.

COMPARISON OF METHODS AND METHODOLOGY SPECIFICITY

The C Reactive Protein Ultrasensitive kit was compared with other commercially available methods for determination of CRP dosage in extremely low concentrations. 07 analyses were performed and the results were evaluated. The linear equation obtained was $Y = 1,036X - 0,122$ with a correlation coefficient equal to 0,9995. With these results we can conclude that the kit has good methodological specificity.

Precision**REPEATABILITY**

The repeatability was calculated from 20 successive determinations, using 3 samples with different concentrations, obtaining the following results:

	Sample 1	Sample 2	Sample 3
Concentration (mg/L)	1,23	2,98	15,02
Standard Deviation (mg/L)	0,01	0,01	0,01
Coefficient of Variation (%)	0,66	0,26	0,07

REPRODUCIBILITY

The reproducibility was calculated from 20 successive determinations for 3 consecutive days, using 3 samples with different concentrations, obtaining the following results:

	Sample 1	Sample 2	Sample 3
Concentration (mg/L)	1,23	2,98	15,03
Standard Deviation (mg/L)	0,01	0,01	0,01
Coefficient of Variation (%)	0,47	0,19	0,04

Sensitivity

Sensitivity was calculated from 20 determinations of a sample free of C-Reactive Protein. The average found was 0,0312 mg/L with standard deviation of 0,001 mg/L. The sensitivity, which indicates the method detection limit, corresponds the average plus 3 times the standard deviation, and is equal to 0,0313 mg/L.

Linearity

The reaction is linear up to the highest point on the curve. For samples with higher values, it is recommended to dilute the sample with NaCl 0,85%, repeat the dosage and multiply the result by the dilution factor.

DIAGNOSTIC SIGNIFICANCE

The levels of C-Reactive Protein (CRP) have been recommended for cardiac risk assessment. Low levels of CRP in apparently healthy patients, may signal coronary heart disease, and thus is an important indicator for AMI and stroke. Importantly, the assessment of CRP as an indicator of cardiac risk can be employed only in the absence of inflammatory conditions, since in those situations, their values are greatly increased.

NUMBER OF TESTS

Variable according to the automated equipment used. Check the number of tests on schedule.

BIBLIOGRAPHIC REFERENCES

- 1- Thompson D, Milford-Ward A, Whicher JT. The value of acute phase protein measurements in clinical practice. *Ann Clin Biochem* 1992;29:123-31.
- 2- Gabay C, Kushner I. Acute-phase proteins and other systemic responses to inflammation. *N Engl J Med* 1999;340:448-54.
- 3- Hansson LO, Lindquist L. C-reactive protein: its role in the diagnosis and follow-up of infectious diseases. *Curr Opin Infect Diseases* 1997; 10:196-201.
- 4- Sipe JD. Acute-phase proteins in osteoarthritis. *Semin Arthritis Rheum* 1995;25:75-86.
- 5- Dati F, Schumann G, Thomas L, Aguzzi F, Baudner S, Bienvenu J et al. Consensus of a group of professional societies and diagnostic companies on guidelines for interim reference ranges for 14 proteins in serum based on the standardization against the IFCC/BCR/CAP reference material (CRM 470). *Eur J Clin Chem Clin Biochem* 1996;34:517-20.
- 6- Schlebusch H, Liappis N, Klein G. High sensitive CRP and creatinine: reference intervals from infancy to childhood. Poster presented at AAC/CSCC; July/August 2001, Chicago, Illinois.

QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone.: +55 (31) 3439.5454 - Fax: +55 (31) 3439.5455
E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

EC	REP	OBELIS S.A.
----	-----	--------------------

Bd. Général Wahis, 53
1030 Brussels, Belgium

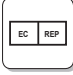
CUSTOMER SERVICE

Customer Advisory Service
Phone.: 0800 0315454
E-mail: sac@bioclin.com.br

ANVISA registration for C Reactive Protein Ultrasensitive kit: 10269360207

Review: March/2013

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MANUFACTURED BY
	BATCH CODE		CONTROL
	DATE OF MANUFACTURE		POSITIVE CONTROL
	USED BY (last day of month)		NEGATIVE CONTROL
	TEMPERATURE LIMITATION (store at)		BIOLOGICAL RISK
	CONTAINS SUFFICIENT FOR <N> TESTS		INFLAMMABLE
	CONSULT INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC DEVICE		POISON
	EUROPEAN AUTHORIZED REPRESENTATIVE		CE MARK
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED